

AdaptDx Tabletop Dark Adaptometer Disinfecting Recommendations

The tabletop AdaptDx® isn't configured to hold any type of breath shield. However, a patient could wear a face mask during the test without interference, but care would need to be taken when patching the non-test eye to ensure a complete light seal.

Whether or not a patient wears a face mask, it is recommended that all patient contact points be wiped with an alcohol pad or disinfectant cloth after each test, and before the next patient. It is a best practice to disinfect the device in front of the patient to reassure them it has been done. Eyepatches should not be reused between patients unless they can be thoroughly disinfected. The bowl of the unit is not a contact point and care should be taken when cleaning so as not to scratch or get liquid into the center lens aperture.

While routine cleaning recommendations for the AdaptDx may be found in the AdaptDx User's Manual Section 7.2, the following steps are recommended after each patient to thoroughly sanitize the AdaptDx:

- All patient contact points (chin rest, forehead rest, and response button) should be swabbed with antibacterial disinfectant or alcohol between patient use.
- Consider cleaning the device in the presence of the next patient to reassure them it has been done.
- Do not immerse any part of the response button in cleaning solution.
- Do not saturate the interior of the viewing tunnel with cleaning solution.
- The red touch screen should be cleaned with a soft, damp antibacterial cloth or pad.
- Do not apply alcohol or a solvent to the touch screen.
- Eyepatches should not be reused between patients unless thoroughly disinfected.



Please contact MacuLogix Customer Service at 1.888.392.6801 with any additional questions.